Edge Stand H-Universal Stand

User Guide

FUJIFILM
Value from Innovation

SonoSite

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

Edge, M-Turbo, MicroMaxx, Titan, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners

Patents: US 8,088,071; US 7,591,786; US 7,534,211; US 6,447,451; CA 2372158; and EP 1589878.

Part Number: P12223-05

Publication Date: June 2018

Copyright © 2018 FUJIFILM SonoSite, Inc. All Rights reserved.

Edge Stand H-Universal Stand

User Guide

Introduction	1
Document conventions	2
Getting Help	3
Stand features	3
Using the stand	5
Inserting and removing the ultrasound system	5
Connecting the ultrasound system	6
Using peripherals and accessories	7
Printer or bar code scanner	7
Triple Transducer Connect	8
Troubleshooting	9
Connectivity diagram	9
Cables and connectors	10
Cleaning and disinfecting	12
Safety	13
Electrical safety	13
Equipment safety	14
Clinical safety	15
Electromagnetic compatibility (EMC)	15
Labeling symbols	15
Labeling symbols	15
Stand dimensions (approximate)	17
Temperature, humidity, and atmospheric pressure limits	17

Introduction

This user guide explains how to use the Edge Stand (used with the SonoSite Edge II, SonoSite Edge, and SonoSite M-Turbo systems) or the H-Universal Stand and peripherals or accessories. For information on a specific peripheral, see the manufacturer's instructions included with that product. For information on the transducers or ultrasound systems, see the ultrasound system user guide.

If you need to disassemble the stand, refer to its assembly instructions available with the stand at www.sonosite.com.

WARNINGS

- ▶ To avoid the risk of patient injury, misdiagnosis, or user injury, read all the warnings in the ultrasound system user guide and supplemental guides.
- ▶ To avoid injury or equipment damage that could result from the stand toppling over, do not apply undue force to the stand. Positioning the stand platform at a low level may help to alleviate this potential hazard.

Document conventions

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ♦.

Symbols and terms used on the system are explained in the ultrasound system user guide.

Getting Help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5589

Printed in the U.S.

Stand features

The stand provides a mobile work platform and a storage area for transducers and other supplies. It also provides connections for accessories and peripherals.

WARNINGS

- ▶ To avoid injury or equipment damage that could result from the stand toppling over, always install the color printer on the front side of the stand, as shown in [Figure 1](#).
- ▶ When installed on the stand, the base of the color printer should not be higher than 50 cm from the floor.

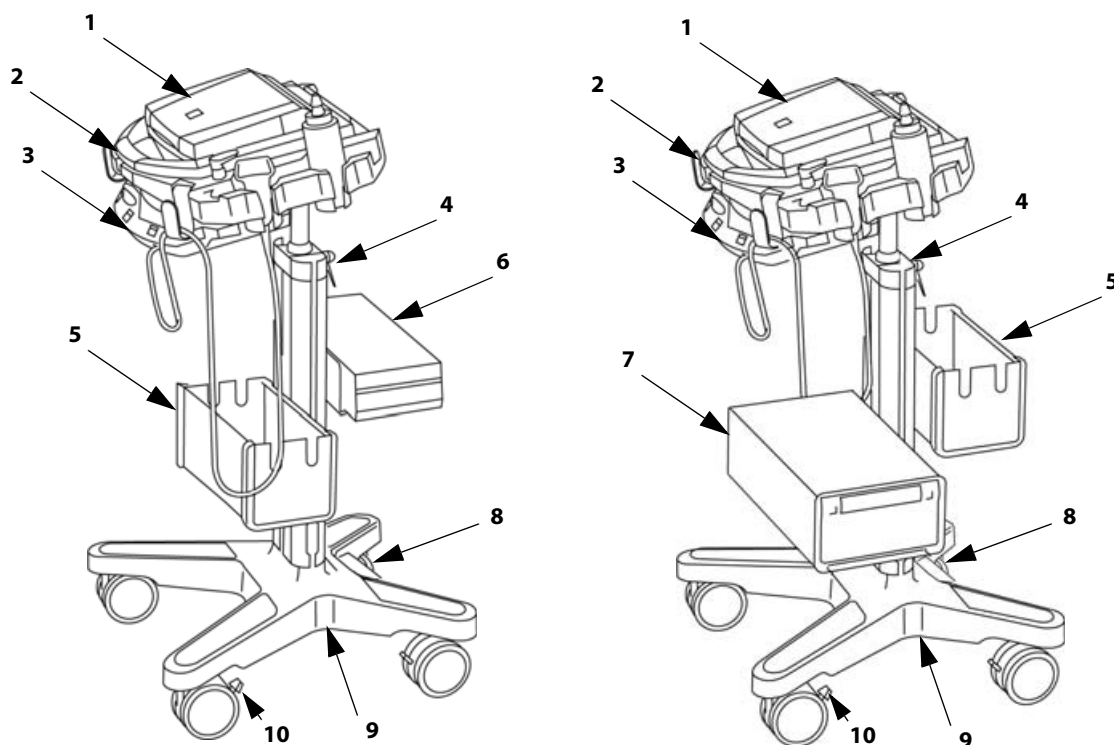
Figure 1 Stand with accessories. The drawing on the left shows the H-Universal Stand with the M-Turbo ultrasound system and optional basket and black-and-white printer. The drawing on the right shows the H-Universal Stand with the M-Turbo ultrasound system and optional basket and color printer.

1	Ultrasound system	6	Black-and-white printer
2	Hook (1 of 4) for cables and other items	7	Color printer
3	Triple Transducer Connect	8	AC adapter 1: Connector for system power cord
4	Height-adjustment lever	9	AC adapter 2: Connector for AC power cord (inside stand base)
5	Basket or storage bin*	10	Locking lever on wheel

*Storage bin is positioned on the opposite side of the printer. Refer to the *Storage Bin Installation Instructions*.

Using the stand

To raise or lower the system platform

- 1 Make sure that the area above the platform is clear.
- 2 Facing the back of the stand, turn the height-adjustment lever counter-clockwise.

The pole may rise automatically from the column.

- 3 Raise or lower the pole to the desired height, and then tighten the lever.

To tighten the lever, turn it clockwise.

If lowering the pole, avoid grasping its base. Skin could get caught and pinched in the gap.

- 4 If you want to reposition the lever (for example, parallel to the pole), pull the lever out, turn it to the desired position, and release.

To lock or unlock a wheel

❖ Push the locking lever on the wheel:

- ▶ Push down to lock.
- ▶ Push up to unlock.

Inserting and removing the ultrasound system

To insert the ultrasound system

- 1 Rotate system latches outward.
- 2 Place the ultrasound system on the system platform, ensuring that the transducer cable is positioned between the handles of the stand.
- 3 Secure by rotating the system latches inward.

To remove the ultrasound system

- 1 Disconnect any cords, cables, or TTC attached to the system.
- 2 Rotate latches outward.
- 3 Lift the ultrasound system.

Connecting the ultrasound system

Attach the mini-dock if you use the system with AC-powered peripherals.

To attach the mini-dock

Note Make sure that the battery is installed before you attach the mini-dock.

- 1 Turn off the ultrasound system.

The mini-dock may not function properly if you connect the system while the power is on.

- 2 Place the mini-dock on the system platform.
- 3 Slide the mini-dock feet under the ultrasound system and firmly press the mini-dock into the I/O connector on the back of the system.

The mini-dock is correctly attached when no gap is between the system and mini-dock.

See [Figure 2](#).

Note If at any time the ultrasound system should lose connectivity, turn off the system and reattach the mini-dock and system.

Always verify that the mini-dock and system are correctly attached after changing a transducer. When you change a transducer, the mini-dock may loosen.

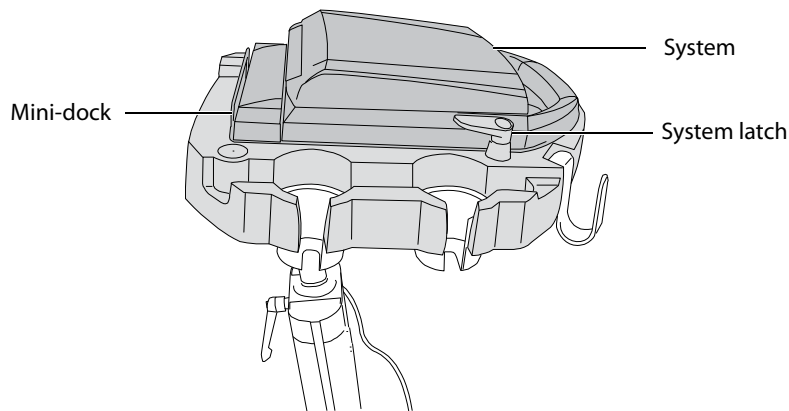
Figure 2 System platform with system and mini-dock attached (H-Universal stand with M-Turbo system shown).

To connect AC power

- 1 If using a mini-dock, ensure that it is connected to the system.
- 2 Connect the cables. See **“Cables and connectors”** on page 10.
 - ▶ DC power cord (A)
 - ▶ System AC power cord (K)

Using peripherals and accessories

Caution

To avoid damaging the system, use only accessories and peripherals recommended by SonoSite with the ultrasound system.

For specific information on using a peripheral, including warnings and cautions, see the manufacturer's operating instructions. For the bar code scanner, see also Bar Code Scanner User Guide.

To connect a peripheral or accessory, see its assembly instructions.

Optional Peripherals and Accessories for Stand

AC-powered peripherals

- ▶ Black-and-white printer
- ▶ Color printer
- ▶ (MicroMaxx, or TITAN) Serial bar code scanner

USB peripherals

- ▶ (SonoSite Edge II, Edge, M-Turbo) USB bar code scanner
- ▶ (SonoSite Edge II, Edge, M-Turbo) USB footswitch

Accessories

- ▶ Mini-dock
- ▶ Triple Transducer Connect

Printer or bar code scanner

You configure the system for the printer or serial bar code scanner on the system's Connectivity setup page. For instructions, see the ultrasound system user guide.

For additional information on the bar code scanner, see the *Bar Code Scanner User Guide*.

To print the screen

- 1 Turn on the printer.
- 2 (Color Printer Only) Change the input setting from S-video to video.
- 3 Do one of the following:
 - ▶ Use the controls on the printer for printing images. Follow the manufacturer's instructions.

- ▶ If the large A or B key (SonoSite Edge II, Edge, M-Turbo) or Delta key (MicroMaxx or *TITAN*) is programmed to print, press it. For instructions to program the key, see the ultrasound system user guide.

You can also save images and print them from the patient list. For instructions, see the ultrasound system user guide.

Triple Transducer Connect

The Triple Transducer Connect (TTC) is an optional three-transducer module that lets you simultaneously connect up to three transducers while the ultrasound system is docked.

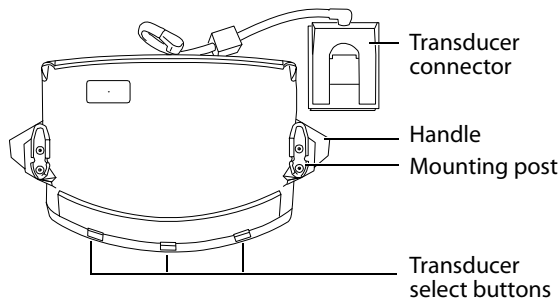
Figure 3 Triple Transducer Connect

To connect a transducer to the TTC

- 1 Pull the transducer latch up and rotate 90°.
- 2 Align and insert the transducer connector to the connector on the bottom of the TTC.
- 3 Turn the latch and press down, securing the transducer connector to the TTC.

To select a transducer

Note

When you use the TTC, there may be a minor reduction of transducer penetration.

- ❖ Push a transducer select button on the TTC.

When selecting another transducer, the system restarts. The light on the button flashes momentarily and then turns solid green.

Only one transducer is accessible at a time.

To remove the TTC

- 1 Remove any attached transducers.
- 2 Disconnect the transducer connector from the system.
- 3 Pull out the knob on each of the two mounting brackets, and slide the TTC out.

Troubleshooting

Symptom	Solution
Printer does not print.	▶ Check the printer selection on the Connectivity setup page. See the ultrasound system user guide. ▶ Check the cable connections. See the printer installation instructions. ▶ Ensure that the printer is turned on and set up properly. See the printer manufacturer's instructions, if necessary.
Stand tips over more easily than expected.	▶ Lower the platform on the stand. ▶ Avoid placing heavy objects on the platform. Too much weight on the platform can cause the stand to become less stable.

Connectivity diagram

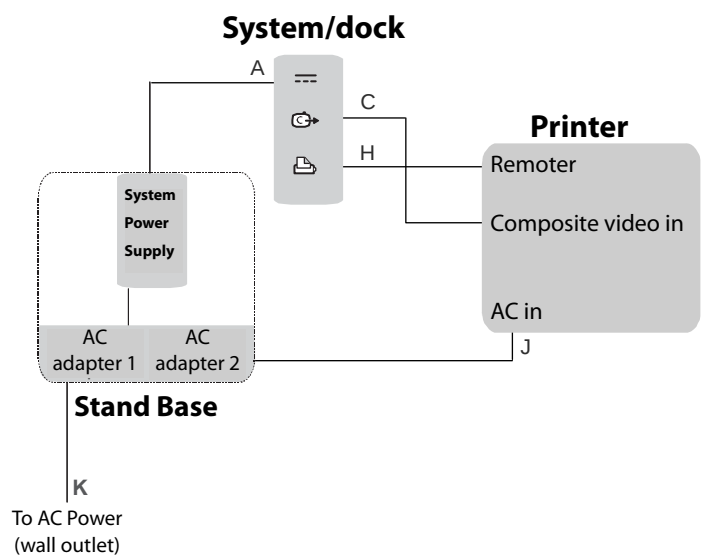
Use the following diagram for assistance disconnecting and reconnecting peripherals. For more detailed information, refer to the assembly instructions. Additional copies are available at www.sonosite.com.

See also **"Cables and connectors"** on page 10.

Caution

To avoid damaging the system, use only accessories and peripherals recommended by SonoSite with the ultrasound system.

Connectivity of printer and system









Cables and connectors

Table 1: Connectivity symbols on mini-docks

Symbol	Definition	Symbol	Definition
	DC input		S-video in
	Print control		DVI video out
	USB		Composite video out
	Ethernet		Audio out
	RS-232 (DVD recorder, VCR, or bar code scanner)		ECG/Footswitch
	S-video out		

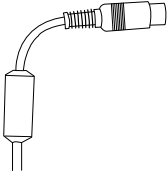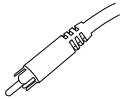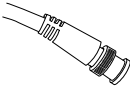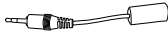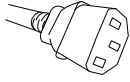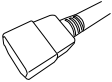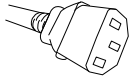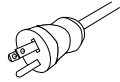Table 2: Cables and connectors

Letter	Name Length	Illustration	Connector 1 [Illustration] [Connects to]	Connector 2 [Illustration] [Connects to]	Peripheral
A	DC power supply cord 6.8 ft./2 m	See <i>“Stand features”</i> on page 3.	 System or mini- dock	Pre-installed	n/a
C	Composite video cable 6 ft./1.8 m		 Mini-dock	 Printer	Printer
H	Print control cable 6 ft./1.8 m		 Mini-dock	 Printer	Printer
J	AC power cord 39 in./1 m		 Printer	 AC adapter 2 on stand base	Printer
K	System AC power cord 10 ft./3 m		 AC adapter 1 on stand base	 Wall outlet	n/a

Cleaning and disinfecting

WARNINGS

- ▶ To avoid electrical shock, before cleaning always disconnect the system from the power supply and remove it from the stand.
- ▶ To avoid injury, always use protective eyewear and gloves when performing cleaning and disinfecting procedures.
- ▶ To avoid infection:
 - ▶ Ensure that the solution expiration date has not passed.
 - ▶ The level of disinfection required for a product is dictated by the type of tissue it contacts during use. Ensure that the solution strength and duration of contact are appropriate for the equipment. For information, see the disinfectant label instructions and the recommendations of the Association for Professionals in Infection Control and Epidemiology (APIC) and FDA.

Caution

Use only recommended cleaners or disinfectants on surfaces. Immersion-type disinfectants are not tested for use on surfaces.

To clean and disinfect a peripheral, refer to its manufacturer's instructions. For the bar code scanner, see also the *Bar Code Scanner User Guide*.

To clean and disinfect the stand or TTC

You can clean and disinfect the exterior surface of the stand or TTC using a recommended cleaner or disinfectant. For the most recent list of approved products, see www.sonosite.com.

- 1 Turn off the ultrasound system and any accessory equipment. Disconnect the system and accessory equipment from AC power.
- 2 Clean the surfaces of the stand to remove any debris or bodily fluids. Use the following procedure:

- a** Use either a pre-moistened wipe or a soft cloth dampened with cleaner or disinfectant.

Approved cleaners and disinfectants for the stand

Cleaner/disinfectant	Minimum wet contact time ¹
SaniCloth AF3 (gray top) ²	3 minutes
SaniCloth Plus (red top)	3 minutes
PI-Spray II	10 minutes
¹ For maximum effectiveness, the component being cleaned must remain wet with disinfectant for a minimum period of time. ² Qualified for use as an intermediate-level disinfectant for mycobacteria. For a more complete list, refer to the cleaning and disinfection tool available at www.sonosite.com/support/cleaners-disinfectants .	

- b** Remove all gel, debris, and bodily fluids from the stand.
- c** With a new wipe, clean all parts of the stand by wiping from the clean areas to the soiled areas. This method helps to avoid cross-contamination.
- d** Observe the minimum wet contact time. Monitor the stand parts for wet appearance. Re-apply with a new wipe if no longer wet.

- 3** Allow the stand to air dry in a clean, well-ventilated space.

For information about cleaning the peripherals, see the manufacturer's instructions.

Safety

Observe the following warnings and cautions before using accessories and peripherals with the ultrasound system and transducers.

Electrical safety

The stand is not an electrical device and does not require ground bond testing. The stand is for facilitating the mobility of peripherals and their connection to the ultrasound system.

WARNINGS

- ▶ To avoid the risk of electrical shock:
 - ▶ Do not plug the power cord of the Edge Stand or H-Universal Stand into a multiple portable socket outlet (MPSO) or use an extension cord.
 - ▶ Use only properly grounded equipment. Shock hazards exist if the power supply is not properly grounded. Grounding reliability can only be achieved when equipment is connected to a receptacle marked "Hospital Only" or "Hospital Grade" or the equivalent. The grounding wire must not be removed or defeated.
 - ▶ Use only accessories and peripherals recommended by SonoSite, including the power supply. Connection of accessories and peripherals not recommended by SonoSite could result in electrical shock. Contact SonoSite or your local representative for a list of accessories and peripherals available from or recommended by SonoSite.
- ▶ To avoid the risk of electrical shock or injury, the equipment shall be connected to a center-tapped single phase supply circuit when users in the United States connect the equipment to a 240V supply system.
- ▶ To avoid the risk of electrical shock and fire hazard, inspect the power supply, plugs, cables, and power cords on a regular basis for damage.

Equipment safety

Cautions

- ▶ Excessive bending or twisting of cables can cause a failure or intermittent operation.
- ▶ Improper cleaning or disinfecting of any part of the accessories and peripherals can cause permanent damage. For cleaning and disinfecting instructions, see **"Cleaning and disinfecting"** on page 12 and the manufacturer's operating instructions.
- ▶ Accessible metal on the signal inputs and outputs on the back and side of the ultrasound system is not protectively earthed. Do not perform a high current ground impedance test involving this part.

Clinical safety

WARNING

To avoid misdiagnosis, do not use non-medical (commercial) grade peripheral monitors. These monitors have not been verified or validated by SonoSite as being suitable for diagnosis.

Electromagnetic compatibility (EMC)

See the safety section of the ultrasound system user guide and supplemental guides for information on electromagnetic compatibility with IEC 60601-1-2, including the Manufacturer's Declaration.

Labeling symbols

The following symbols are used on the products, packaging, and containers.

Symbol	Definition
	Follow instructions for use
	Reading the accompanying documentation is necessary for safe operation of the medical device
	Manufacturer, or Manufacturer and date of manufacture
	Authorized representative in the European Community
	Class 1 device indicating manufacturer's declaration of conformance with Annex VII of 93/42/EEC
	Catalog number
	Collect separately from other household waste (refer to European Commission Directive 93/86/EEC). Refer to local regulations for disposal.

Symbol	Definition
	Do not get wet
	Do not stack over # high, where # represents the number on the label
	Fragile
	Paper recycle
	Serial number type of control number
	Temperature limitation
	Atmospheric pressure limitation
	Humidity limitations
	Caution
	Total equipment mass

Specifications

For information on the peripherals, see the manufacturer's specifications.

Stand dimensions (approximate)

Width: 20.25 in. (51.4 cm)

Depth: 22 in. (55.9 cm)

Height: 45 in. (114.3 cm) maximum, 33 in. (83.8 cm) minimum

Height range: 12 in. (30.5 cm)

Weight: 40.0 lbs. (18.1 kg)

Safe working load per storage bin: 5 kg

Total stand weight with system and peripherals: 46 kg maximum

Temperature, humidity, and atmospheric pressure limits

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