SONOSITE

Sonosite Mini-Dock User Guide

| Manufacturer | EC Authorized Representative | Australia Sponsor |
|--|--|--|
| FUJIFILM SonoSite, Inc. 21919 30th Drive SE Bothell, WA 98021 USA T: 1-888-482-9449 or 1-425-951-1200 F: 1-425-951-1201 | FUJIFILM SonoSite B.V. Joop Geesinkweg 140 1114 AB Amsterdam, The Netherlands | FUJIFILM SonoSite Australasia Pty Ltd 114 Old Pittwater Road BROOKVALE, NSW, 2100 Australia |

CAUTION

United States federal law restricts this device to sale by or on the order of a physician.

SONOSITE, M-Turbo, Edge, MicroMaxx, and TITAN are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation. All other trademarks are the property of their respective owners.

Patent: www.sonosite.com/patents

P05403-07

Published Sep 30, 2022

Copyright © 2022 FUJIFILM SonoSite, Inc. All rights reserved.

Sonosite Mini-Dock User Guide

| | |
|--|---|
| Introduction | 1 |
| Document conventions | 1 |
| Getting help | 1 |
| Attaching and removing the mini-dock | 2 |
| Mini-dock connectors | 3 |
| Cleaning and disinfecting | 4 |

Introduction

The Sonosite mini-dock provides connections for accessories and peripherals on any of the following ultrasound systems:

- Sonosite Edge II ultrasound system
- Edge ultrasound system
- M-Turbo ultrasound system
- MicroMaxx ultrasound system
- *TITAN* ultrasound system

NOTE

Each ultrasound system uses a different model mini-dock.

Document conventions

The document follows these conventions:

- A **WARNING** describes precautions necessary to prevent injury or loss of life.
- A **CAUTION** describes precautions necessary to protect the products.
- A **NOTE** provides supplemental information.
- Numbered and lettered steps must be performed in a specific order.
- Bulleted lists present information in list format but do not imply a sequence.

Symbols and terms used on the system and transducers are explained in the ultrasound system user guide.

Getting help

- FUJIFILM Sonosite Technical Support:

| | |
|--------------------------|---|
| United States and Canada | +1 877-657-8118 |
| Europe and Middle East | Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 |

| | |
|------------------|--|
| | German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451 |
| Asia and Pacific | +61 2 9938 8700 |
| Other regions | +1 425-951-1330 or call your local representative |
| Fax | +1 425-951-6700 |
| Email | General: ffss-service@fujifilm.com United Kingdom: uk-service@fujifilm.com Europe, Middle East, and Africa: eraf-service@fujifilm.com Asia and Pacific: ffss-apacme-service@fujifilm.com |
| Web | www.sonosite.com |

Printed in the U.S.

Attaching and removing the mini-dock

WARNING

To avoid the risk of electrical shock to the patient, do not touch the ungrounded connectors on the back of the mini-dock while simultaneously touching a patient.

CAUTION

- To avoid damaging the mini-dock, do not perform high current grounding impedance testing involving this part. Accessible metal of the mini-dock is not protectively earthed.
- To avoid damaging equipment, remove mini-dock from system prior to transporting.


1. Ensure that the ultrasound system is turned off.
Connecting the system and mini-dock while the power is on may cause the mini-dock to function improperly.
2. Slide the mini-dock feet under the ultrasound system, and firmly press the mini-dock into the system connector. The mini-dock is correctly attached when no gap is between the system and mini-dock.

NOTE

If at any time the ultrasound system loses connectivity, turn off the system and reattach the mini-dock and system. Always verify that the mini-dock and system are correctly attached after changing a transducer. When you change a transducer, the mini-dock may loosen.

3. To remove the mini-dock, grasp the handle on the ultrasound system, push back on the release lever, and then pull the ultrasound system forward.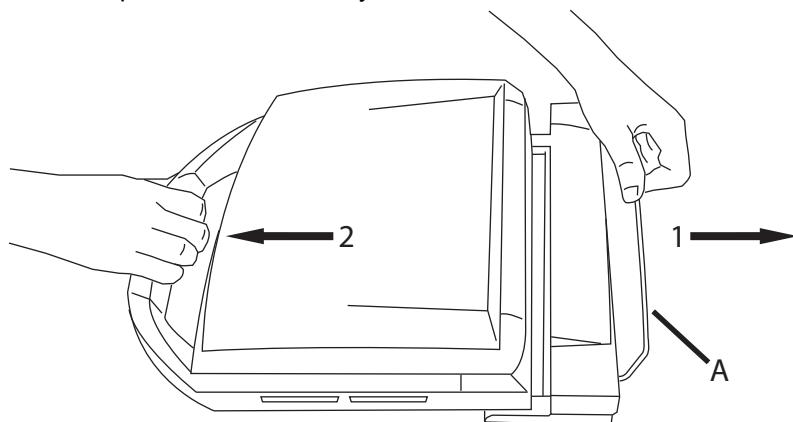

A. Release lever

Mini-dock connectors

Each connector on the mini-dock has a symbol that describes its use. The specific set of connectors depends on the version of mini-dock.

Table 1. Connectivity symbols on mini-dock with printer compatibility

| Symbol | Definition | Symbol | Definition |
|--------|--|--------|---------------------|
| | DC input | | S-video in |
| | Print control | | DVI video out |
| | USB | | Composite video out |
| | Ethernet | | Audio out |
| | RS-232 (DVD recorder, VCR, or barcode scanner) | | ECG/Footswitch |
| | S-video out | | |

Table 2. Connectivity symbols on mini-dock without printer compatibility

| Symbol | Definition | Symbol | Definition |
|--------|--------------------------|--------|----------------|
| | DC input | | S-video in |
| | USB | | DVI video out |
| | Ethernet | | Audio out |
| | RS-232 (barcode scanner) | | ECG/Footswitch |

Cleaning and disinfecting

CAUTION

Use only approved cleaners or disinfectants on surfaces.

Immersion-type disinfectants are not approved for use on surfaces. For approved cleaners and disinfectants, refer to the cleaners and disinfectants tool available at www.sonosite.com/support/cleaners-disinfectants.

The exterior surface of the mini-dock can be cleaned and disinfected with an approved cleaner or disinfectant. For cleaning and disinfecting instructions, see your system's user guide, which is available at www.sonosite.com/support/user-documents.

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