Sonosite Ultrasound System

User Guide Errata

Manufacturer	EC Authorized Representative	Australia Sponsor
FUJIFILM SonoSite, Inc.	FUJIFILM SonoSite B.V.	FUJIFILM SonoSite Australasia Pty Ltd
21919 30th Drive SE	Joop Geesinkweg 140	114 Old Pittwater Road
Bothell, WA 98021 USA	1114 AB Amsterdam,	BROOKVALE, NSW, 2100
T: 1-888-482-9449 or 1-425-951-1200	The Netherlands	Australia
F: 1-425-951-1201		

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite, SONOSITE, SonoSite SII, M-Turbo, and SonoSite X-Porte are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P23353-01

Publication date: July 2021

Copyright © 2021 FUJIFILM SonoSite, Inc. All rights reserved.

Sonosite Ultrasound System User Guide Errata

Introduction	1
Document conventions	1
Getting Help	2
Cleaning and Disinfecting	2
Cleaning Accessories	2
Safety	3

Introduction

The content in this document corrects the following user guides. The revisions will be made in the next update.

- ▶ Sonosite Edge II ultrasound system
- ▶ Sonosite SII ultrasound system
- ▶ Sonosite iViz ultrasound system
- ▶ Sonosite M-Turbo ultrasound system
- ▶ Sonosite X-Porte ultrasound system

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ♦.

For a description of labeling symbols that appear on the product, see “Labeling Symbols” in the user guide.

Getting Help

FUJIFILM Sonosite Technical Support

United States or Canada	+1 (877) 657-8118
Europe and Middle East	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia and Pacific	+61 2 9938 8700
Other regions	+1 (425) 951-1330, or call your local representative
Fax	+1 (425) 951-6700
E-mail	General: ffss-service@fujifilm.com Europe, Middle East, and Africa: eraf-service@fujifilm.com Asia and Pacific: ffss-apacme-service@fujifilm.com
Web	www.sonosite.com

Cleaning and Disinfecting

All references to blood and bodily fluids have been removed from this chapter.

The following note has been added to this chapter:

Note

FUJIFILM Sonosite defines ultrasound guided percutaneous procedures as non-critical use applications. Following percutaneous procedures, users should clean and disinfect transducers for non-critical uses as outlined in this user guide.^a

- a.“Disinfection of Ultrasound Transducers Used for Percutaneous Procedures. Intersocietal Position Statement.” *American Institute of Ultrasound in Medicine / J Ultrasound Med.* February 16, 2021; 9999: p. 1–3.

Cleaning Accessories

To clean and disinfect the ECG leadwires, refer to the manufacturer’s instructions.

Safety

The following has been added to the safety chapter.

Please report any serious safety incident that occurs in relation to the ultrasound system to FUJIFILM Sonosite and to the competent authority of the country in which the user and patient are located.

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